

## StatStrip Competency Post Test

Name\_\_\_\_\_

Unit\_\_\_\_\_

Date\_\_\_\_\_

Score\_\_\_\_\_

1. Once opened, test strips are good for 180 days and control solutions are good for 90 days or until manufacturer's printed expiration date whichever comes first.

☐ True    ☐ False

2. Comments are mandatory when documenting:

- A. Action taken when QC results are out of range
- B. Action taken when critical patient results are obtained
- C. All of the above

3. Once opened, control solutions and test strips must be dated with open and expiration date.

☐ True    ☐ False

4. Which of the following are true about cleaning the meter?

- A. Clean/disinfect the meter surface with locally approved disinfectant wipes (I. E. Sani Wipes) between every patient
- B. Clean/disinfect the meter surface with 10% bleach wipes (i.e. Clorox wipes) for isolation and C. Diff. patients
- C. All of the above

5. The test strip must fill completely upon touching the blood droplet. IF the test strip does not fill completely, do not touch the test strip to the blood droplet a second time. Discard the test strip and repeat the test with a new strip.

☐ True    ☐ False